ENDOCRAFT

P.O. Box 8672, 25 New Chardon Street, Boston, MA 02114

Zeitels Vocal Fold Infusion Needle

**Aguja para infusión de cuerdas vocales Zeitels
Aiguille à perfusion pour cordes vocales Zeitels
Ago per infusione nelle corde vocali Zeitels
Stimm lippen-Infusionsnadelsatz nach Zeitels**

Instructions for Use

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Mode d'emploi

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ENGLISH:**Indications:**

- **Microlaryngoscopic subepithelial infusion of saline and epinephrine into the true vocal fold phonatory mucosa.**
- **Microlaryngoscopic subepithelial infusion of fluid steroids into the true vocal fold phonatory mucosa.**
- **Microlaryngoscopic subepithelial infusion of Avastin (bevacizumab) solution into the true vocal fold phonatory mucosa.**
- **Microlaryngoscopic subepithelial infusion of Cidofovir solution into the true vocal fold phonatory mucosa.**

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ENGLISH:Endocraft**Zeitels Vocal Fold Infusion Needle Instructions For Use****Limited Warranty**

Endocraft warrants the original purchaser that the equipment shall be free from defects in material and workmanship for a period of one [1] year from the date of shipment. Endocraft's obligation under this warranty shall be limited to repair or replacement, at the option of Endocraft. The above warranty is contingent upon normal usage and does not cover products that have been modified without Endocraft's approval or which have been subjected to non-recommended use or unusual physical or chemical stress.

Notices

No part of this document may be reproduced, stored in a retrieval system, or transmitted, in any form or means -- electronic, mechanical, photocopying, recording, or otherwise -- without the expressed, written consent of Endocraft. Additional copies of this document can be ordered from Endocraft.

Manufacturer**Endocraft**

**P.O. Box 8672
25 New Chardon Street
Boston, MA 02114
USA**

Web: www.endocraft-llc.com

All inquiries, repairs, and/or returns should be directed to the address below:

Endocraft

**89 Bellows Street
Warwick, RI 02888
USA**

Phone: (401) 369-7344

Fax: (401) 228-7397

Email: info@endocraft-llc.com

*** WARNING: THE VOCAL FOLD INFUSION NEEDLE SHOULD ONLY BE USED FOR INJECTING FLUIDS INTO THE PHONATORY MUCOSA. THE NEEDLE SHOULD NOT BE USED FOR INJECTING OTHER SITES AND LOCATIONS OR INJECTING VISCOUS MATERIALS.**

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SPANISH / ESPAÑOL:

Description and Overview:

The Vocal Fold infusion Needle was designed for single use for the primary purpose of: 1. placing fluid medications in the phonatory mucosa or 2. performing saline hydrodissection, which can enhance precision of delicate dissection within the layered microstructure of the vocal fold. Infusion of saline with or without epinephrine into the superficial lamina propria (SLP) of the phonatory mucosa can facilitate the surgeon's ability to maximally preserve the laminae propria. This is accomplished in a number of ways:

- Increasing the volume of the SLP helps preserve it during tangential dissection.
- Volume expansion of the SLP places tension on the epithelium, which facilitates a clean incision [epithelial cordotomy].
- Vasoconstricting the microcirculation by means of epinephrine as well as hydrostatic pressure enhances visualization of tissue planes.
- Distending the normal SLP enhances visual and palpable discrimination between normal and pathological tissue.
- If a laser is used, the infusion fluid acts as a heat sink to prevent unnecessary trauma to the SLP.

Indications:

- Microlaryngoscopic subepithelial infusion of saline and epinephrine into the true vocal fold phonatory mucosa.
- Microlaryngoscopic subepithelial infusion of fluid steroids into the true vocal fold phonatory mucosa.
- Microlaryngoscopic subepithelial infusion of Avastin (bevacizumab) solution into the true vocal fold phonatory mucosa.
- Microlaryngoscopic subepithelial infusion of Cidofovir solution into the true vocal fold phonatory mucosa.

Infusion and Dissection of the Vocal Fold:

If dissection is performed in the superficial lamina propria (SLP), cold instruments facilitate precise tangential dissection around the curving fold. This allows for maximal preservation of the SLP, and for pliability of the regenerating epithelium.

The subepithelial saline-epinephrine infusion into the SLP (Reinke's space) (figures 1, 2) improves assessment of lesion depth prior to excision. It can also help define areas of scarred phonatory mucosa in which the epithelium is adherent to the vocal ligament. If a cancer has invaded the vocal ligament, the SLP at the perimeter of the lesion will distend, creating a contour depression in the region of the cancer.

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SPANISH / ESPAÑOL:

ENGLISH:

The subepithelial infusion assists with the surgeon's precise technical execution of the surgery in a number of other ways:

1. The infusion facilitates mucosal incisions by improving visualization of the lateral border of the lesion and by distending the SLP so that the overlying epithelium is under tension.
2. The infusion also increases the depth of the SLP, which facilitates less traumatic dissection in this layer and leads to regenerated epithelium that is more pliable.
3. The epinephrine and hydrostatic pressure of the infusion vasoconstrict microcirculation in the SLP and improves both visualization and precise dissection.
4. If a laser is used, the saline acts as a heat sink, which decreases the potential of thermal trauma to normal vocal fold tissue.

Figure 1: The subepithelial infusion distends the SLP and lifts the lesion. This enhances the precision of the procedure and facilitates maximal preservation of the vocal fold's layered microstructure.

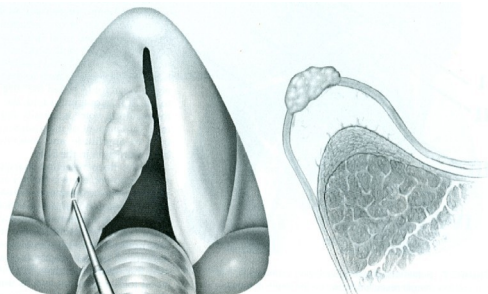
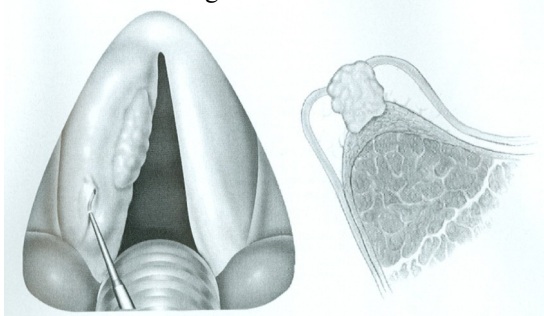
Figure 2: The subepithelial infusion reveals the tumor invasion into the vocal ligament.

**References:**

1. Zeitels, S.M., Vaughan, C.W., *A submucosal vocal fold infusion needle*. Otolaryngology: Head and Neck Surgery, 1991, 105: p 478-479.
2. Zeitels, S.M., *Premalignant epithelium and microinvasive cancer of the vocal fold: The evolution of phonomicrosurgical management*. Laryngoscope, 1995. Supplement 67: p. 1-51

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**Referencias:**

1. Zeitels, S.M., Vaughan, C.W., *A submucosal vocal fold infusion needle*. Otolaryngology: Head and Neck Surgery, 1991, 105: p 478-479.
2. Zeitels, S.M., *Premalignant epithelium and microinvasive cancer of the vocal fold: The evolution of phonomicrosurgical management*. Laryngoscope, 1995. Supplement 67: p. 1-51

SPANISH / ESPAÑOL:

ENGLISH:

3. Zeitels, S.M., Phonomicrosurgical Treatment of early Glottic Cancer and Carcinoma in Situ. American Journal of Surgery, 1996. 172: p. 704-709.
4. Kass, E.S., Hillman, R.E., Zeitels, S.M., The Submucosal Infusion Technique in Phonomicrosurgery. Annals of Otology, Rhinology, & Laryngology, 1996. 105: p 341-347
5. Zeitels, S.M., *Atlas of Phonomicrosurgery and Other Endolaryngeal Procedures for Benign and Malignant Disease*. 2001, San Diego: Singular
3. Zeitels, S.M., Phonomicrosurgical Treatment of early Glottic Cancer and Carcinoma in Situ. American Journal of Surgery, 1996. 172: p. 704-709.
4. Kass, E.S., Hillman, R.E., Zeitels, S.M., The Submucosal Infusion Technique in Phonomicrosurgery. Annals of Otology, Rhinology, & Laryngology, 1996. 105: p 341-347
5. Zeitels, S.M., *Atlas of Phonomicrosurgery and Other Endolaryngeal Procedures for Benign and Malignant Disease*. 2001, San Diego: Singular

How Supplied:

The Zeitels Vocal Fold Infusion Needle is supplied in a non-sterile pouch.

WARNING: Product is provided non-sterile. Cleaning and sterilizing is required before use.

WARNING: Recommended use is one (1) time.

CAUTION: Product should not be used if any portion of the needle has been bent, barbed, or altered from its original conformation, especially at the distal 25-gauge penetrating tip.

CAUTION: Patency of the needle should be tested with sterile saline prior to use and then cleared by insufflating air. Product should be discarded if it is determined to be partially or completely obstructed. Under no circumstances should the needle be manipulated or instrumented, (such as with a stylet), to re-establish patency

Cleaning and Sterilization of the Infusion Needle Set:

Prior to sterilization, the infusion needles should be irrigated with 30 cc of saline or sterile water. Endocraft recommends 100% ethylene oxide (EtO) gas sterilization for all components. Sterilization parameters are specified below.

EtO Sterilization (100%EtO):

Concentration: 725 mg/L (\pm 25 mg/L)
 Temperature: 130°F (10°F/-5°F) / 54.4°C
 Exposure Time: 2 Hours (+10 minutes)
 Humidity: 50% (+30%/-10%)

Steam Sterilization:

Temperature: 270 F
 Exposure Time: 4 minutes
 Cycle: Pre-vacuum

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ENGLISH:**STERRAD® 100 Sterilization System¹:**

Vacuum Phase	Evacuation of sterilization chamber to 300 mTorr pressure	5-20 min.
Injection Phase	Injection of 1.8ml aqueous solution of H ₂ O ₂ and vaporization	6 min.
Diffusion Phase	Diffusion of H ₂ O ₂ in chamber and throughout load	44 min.
Plasma Phase	Low-temperature, gas plasma with 400W power at 500 mTorr pressure	15 min.
Vent Phase	Return of chamber to atmospheric pressure	5 min.
Total Cycle Time		approx. 75 min.

¹ STERRAD is a registered trademark of Johnson & Johnson Medical, Inc.

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Bibliografia:

1. Zeitels, S.M., Vaughan, C.W., *A submucosal vocal fold infusion needle*. *Otolaryngology: Head and Neck Surgery*, 1991, 105: p 478–479.
2. Zeitels, S.M., *Premalignant epithelium and microinvasive cancer of the vocal fold: The evolution of phonomicrosurgical management*. *Laryngoscope*, 1995.

Supplement 67: p. 1–51.

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1. Zeitels, S.M., Vaughan, C.W., *A submucosal vocal fold infusion needle*. Otolaryngology: Head and Neck Surgery, 1991, 105: p 478–479.
2. Zeitels, S.M., *Premalignant epithelium and microinvasive cancer of the vocal fold: The evolution of phonomicrosurgical management*. Laryngoscope, 1995. Supplement 67: p. 1–51.
3. Zeitels, S.M., Phonomicrosurgical Treatment of early Glottic Cancer and Carcinoma in Situ. American Journal of Surgery, 1996. 172: p. 704–709.
4. Kass, E.S., Hillman, R.E., Zeitels, S.M., The Submucosal Infusion Technique in Phonomicrosurgery. Annals of Otolaryngology, Rhinology, & Laryngology, 1996. 105: p 341–347.
5. Zeitels, S.M., *Atlas of Phonomicrosurgery and Other Endolaryngeal Procedures for Benign and Malignant Disease*. 2001, San Diego: Singular.

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ITALIAN / ITALIANO:

3. Zeitels, S.M., Phonomicrosurgical Treatment of early Glottic Cancer and Carcinoma in Situ. American Journal of Surgery, 1996. 172: p. 704–709.
4. Kass, E.S., Hillman, R.E., Zeitels, S.M., The Submucosal Infusion Technique in Phonomicrosurgery. Annals of Otolaryngology, Rhinology, & Laryngology, 1996. 105: p 341–347.
5. Zeitels, S.M., *Atlas of Phonomicrosurgery and Other Endolaryngeal Procedures for Benign and Malignant Disease*. 2001, San Diego: Singular.

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